



Doc No.: RP-RA-04 RE03

Instructions For Use

This set of instructions was written with respect to the following medical devices:

- Virox Multi-Surface Disinfecting Wipes 100s (Macerator Friendly) (Product code: VR3939)
- Virox Multi-Surface Disinfecting Wipes 200s (Alcohol-free) (Product code: VR3936)
- Virox Hard Surface Bactericidal Wipes 200s (Product code: VR3938)
-

The Basic UDI-DI of these medical devices is 88863897ViroxV5.

Our intended use of these products is to kill 99.99% of germs. It is designed for disinfection and cleaning of hard surfaces. Not for end-point sterilization. Our indications of use are for disinfection of non-invasive medical devices.

These products are designed for disinfection of all surfaces, such as metal, rubber and fabrics, and are meant to be disposed after single use. Avoid acrylic glass, acrylic surfaces and other plastic surfaces when using alcohol-based wipes.

The device is a medical device.

To use this product, turn the cap counter clockwise to remove it from the canister. Remove the freshness seal and locate the first wipe at the centre of the roll. Push the corner of the first wipe through the opening in the cap. (Figure 1) Close the cap after use to keep wipes moist. Ensure the lid is firmly shut after each use to prevent the wipes from drying.

DIRECTIONS FOR USE:

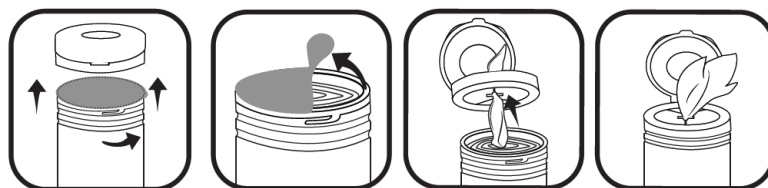


Figure 1: Diagram of directions for use

The wipes can be used on general non-invasive medical devices as depicted in Figure 2, such as hospital beds, wheelchairs and thermometers.



Figure 2: Types of non-invasive medical devices that are suitable for wet wipes

Make sure that all visible soiling on target surface is removed before disinfection. Wipe surface thoroughly with the impregnated wipe, ensure complete wetting and allow the surface to dry completely. Duration of using wet wipes is dependent on the drying time. Ensure that the antimicrobial efficiency of the product matches its intended use. Not for end-point disinfection of semi-critical and critical medical devices.

As a precaution, this product is for external use only and users should keep out of reach of children. This product is flammable so it should be kept away from heat, sparks and open flame. Avoid contact with eyes. Store at room temperature. Wearing of gloves (eg. Nitrile rubber) is recommended. First aid: if in contact with eyes; hold eyes open and rinse slowly and gently with water for 15-20 minutes. If contact lenses are present, remove them immediately and rinse eyes. Call a poison control center for treatment advice. Any serious incident that has occurred in relation to the device should be reported to manufacturer and local authority of the country.

To store this, keep canisters tightly closed in a dry, cool and well-ventilated place. Store in a dry place no lower in temperature than 10°C or higher than 50°C. Do not contaminate water, food or feed by storage or disposal. Dispose of wipe in trash after use. Do not flush. Non-refillable container. Do not reuse or refill this empty canister. Offer empty canister for recycling. If recycling is not available, discard canister in trash.

