



Doc No.: RP-RA-02 RE03

## Instructions For Use

This set of instructions for use was written with respect to following non-invasive medical devices:

- Virox Multi-surface Disinfecting Wipes 60s (Product code: VR3885)
- Virox Hard Surface Bactericidal Wipes 100s (Product code: VR3886)

The Basic UDI-DI of these non-invasive medical devices is 88863897ViroxV5.

Our intended use of these products is to kill 99.99% of germs. It is designed for disinfection and cleaning of hard surfaces. Not for end-point sterilization. Our indications of use are for disinfection of non-invasive medical devices.

These products are designed for disinfection of all surfaces, such as metal, rubber and fabrics, and are meant to be disposed after single use. Avoid acrylic glass, acrylic surfaces and other plastic surfaces when using alcohol-based wipes.

The device is a medical device.

This product was meant for external use only. Store in a cool place, away from naked flame and direct sunlight. Do not flush.

Open the lid upwards to remove wipes. Pull out one sheet and wipe desired surface for disinfection. Close lid and reseal label after use to retain moisture in the pack. Discard properly after use. (Figure 1)

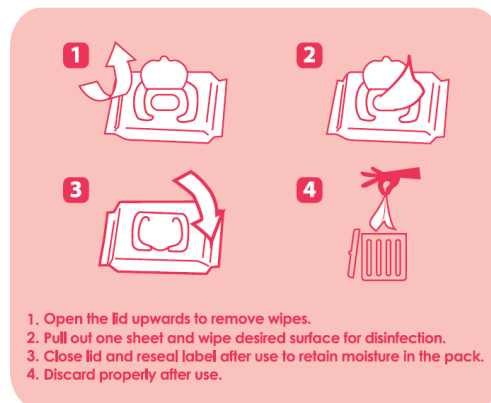


Figure 1: Diagram of usage directions

The wipes can be used on general non-invasive medical devices as depicted in Figure 2, such as hospital beds, wheelchairs and thermometers.



Figure 2: Types of non-invasive medical devices that are suitable for wet wipes

Make sure that all visible soiling on target surface is removed before disinfection. Wipe surface thoroughly with the impregnated wipe, ensure complete wetting and allow the surface to dry completely. Duration of using wet wipes is dependent on the drying time. Ensure that the antimicrobial efficiency of the product matches its intended use. Not for end-point disinfection of semi critical and critical medical devices.

As a precaution, users should keep this product out of reach of children. Keep away from heat, sparks and open flame. Avoid contact with eyes. Store at room temperature. Wearing gloves (eg. nitrile rubber) is recommended. First aid: if in contact with eyes; hold eyes. Open and rinse slowly and gently with water for 15-20 minutes. If contact lenses are present, remove them immediately and rinse eyes. Call a poison control center for treatment advice. Any serious incident that has occurred in relation to the device should be reported to manufacturer and local authority of the country.

