



Doc No.: RP-RA-02b RE01

## Instructions For Use

This set of instructions for use was written with respect to following medical devices:

- Schulke Mikroqid Universal Wipes 120s (Product code: SK2033)

The Basic UDI-DI of this device is 88863109SchulkeCJ.

This device (Mikroqid Universal Wipes) is disinfecting, bactericidal and Isopropyl alcohol wipes which comes in the form of soft pack.

Mikroqid® universal wipes are ready to use disinfectant wipes soaked with an active solution especially developed for cleaning and rapid disinfection of hard surfaces and sensitive surfaces of medical devices. For disinfection of invasive medical devices, as end-point of processing. This product is not for cleaning edibles or food contact surfaces.

Mikroqid® Universal Wipes are effective against bacteria including *S. aureus*, *E. Hirae* and *P. aeruginosa*, *S. aureus* (MRSA), *E. faecium* (VRE), *K. pneumonia* (CRE) and *A. baumannii* (MRAB), yeast including *C. albicans* and *C. auris*, Viruses including Polyoma Sv40, SARs-CoV-2 (COVID-19), Rotavirus, Norovirus (MNV Surrogate), Monkeypox (MVA surrogate), Adenoviruses and Tuberculocidal, including *M. terrae*.

This product is suitable for material-sparing, uncomplicated quick disinfection of surfaces of all kinds including sensitive surfaces, e.g. of makrolon, acrylic glass, polysulfone as well as medical surfaces such as displays, keyboards, control panels and touchscreens of sensitive medical equipment.

These products are designed for disinfection of all surfaces, such as metal, rubber and fabrics, and are meant to be disposed after single use.

The device is a medical device.

This product was meant for external use only. Store in a cool place, away from naked flame and direct sunlight. Protect from frost, heat and sunlight. Do not flush.

Open the lid upwards to remove wipes. Pull out one sheet and wipe desired surface for disinfection. Close lid and reseal label after use to retain moisture in the pack. Discard properly after use. (Figure 1)

How to use:



Figure 1: Diagram of usage directions

Make sure that all visible soiling on target surface is removed before disinfection. Wipe surface thoroughly with the impregnated wipe, ensure complete wetting and allow the surface to dry completely. Duration of using wet wipes is dependent on the drying time. Ensure that the antimicrobial efficiency of the product matches its intended use.

As a precaution, users should keep this product out of reach of children. Keep away from heat, sparks and open flame. Avoid contact with eyes. Store at room temperature in the original container. Only recycle completely emptied packaging. Leftover product can be disposed of with general waste. The suitability of the spectrum of action is to be tested for the intended application. First aid: if in contact with eyes; hold eyes. Open and rinse slowly and gently with water for 15-20 minutes. If contact lenses are present, remove them immediately and rinse eyes. Call a poison control center for treatment advice. Any serious incident that has occurred in relation to the device should be reported to manufacturer and local authority of the country.