



virox

Alcohol Free Multi-Surface Disinfectant Liquid Spray

For rapid disinfection of most surfaces including those sensitive to alcohol.
Effective in killing Influenza A H1N1 Virus



For more information, please call +65 - 6542 2100 or email to enquiries@kleen-pak.com

 Virox  www.kleen-pak.com

Overview

Virox Alcohol Free Multi-Surface Disinfectant Liquid Spray enables a rapid disinfection of most surfaces such as metal, rubber, acrylic, plastic, fabrics, and those sensitive to alcohol.

Specifically formulated for the control of cross-contamination and uses alcohol-free formula. Ideal to use on materials that are sensitive to alcohol.

Virox Alcohol Free Multi-Surface Disinfectant Liquid Spray is great for use in cleaning, decontamination and disinfecting all surfaces.

Active Ingredients

Benzalkonium Chloride (0.24%), N-(3-Amino Propyl)-N-dodecylpropane-1, 3-diamine (0.08%).

Antimicrobial Effectiveness Tested

Test Organism	EN Method	Results (Percentage reduction) Contact time 5 mins
Escherichia coli	EN 13727:2012	>99.999%
Candida albicans	EN 13624:2013 and EN 16615:2015	>99.99%
Staphylococcus aureus Pseudomonas aeruginosa Enterococcus hirae	EN 13727:2012 and EN 16615:2015	>99.999%
Salmonella typhimurium	EN 1276	>99.999%
MRSA	EN 1276	>99.999%
Streptococcus pyrogenes	EN 1276	>99.999%
Bacillus subtilis	EN 1276	>99.999%
SARS-CoV-2	EN 14476	>99.99%
Influenza A H1N1 Virus	EN 14476	>99.99%

Specifications

Product Code	Volume	Bottle Spray Dimensions	EAN Barcode	Units / Carton	Carton Size & Weight	Pallet Weight	Pallet Height	Cartons / Pallet	Cartons / Layer	Layers / Pallet
VR4001	1L	D95mm x H290mm	8886389740018	6	L300 x W200 x H300 5.55kg	576	1650	100	20	5



Suitable for disinfection of non-invasive medical devices in all areas and hard surfaces such as medical equipment surfaces, patient treatment areas, operating tables and similar work surfaces

Features and Benefits



Kills 99.9% of germs in 1 minute



Designed for all surface disinfecting and personal hygiene



Fragrance-Free



Manufactured under ISO 9001, 22716 & 13485 quality system to ensure complete quality control




Alcohol-Free



Formulated in Singapore

Directions For Use

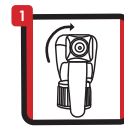

Turn nozzle to ON position



Keep the bottle upright, spray from 20-25cm away



Wipe over with clean damp cloth and allow to dry.